Technisches Datenblatt (TDB) / Technical Data Sheet (TDS)

PPERP X2 Partikelfiltrierende Halbmaske FFP2 NR / PPERP X2 Particle Filtering Half Mask FFP2 NR

Produktinformationen / Product information

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PPERP X2 is an ergonomically shaped, disposable particle filtering half masks made of 3 ultrasonic welded parts - upper, middle, and lower part. Each part consists of polypropylene layers including a melt blown filter layer. Two elastic headbands, an adjustable nose wire and the large contact area of the lower part ensure good sealing and high comfort. After correct positioning PPERP X2 remains in place the foreseeable period of use.

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Atemwiderstand / Breathing resistance

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INSTRUCTIONS FOR USE

PPERP X2 (hereafter also referred to as "mask") is approved as personal protection equipment (PPE) according to European Regulation (EU) 2016/425 and meets the requirements of EN 149:2001+A1:2009. This mask should only be used to protect its wearer from solid particles and non-volatile liquid aerosols. The mask does not protect against gases and vapours.

Please make sure to observe all warnings.

MARKING

- NR** Non re-usable, for single use only, use only for one shift
- End of shelf life, do not use after date format: YYYY/MM
- Temperature range of storage conditions in °C
- Maximum relative humidity of storage conditions in %
- Follow instructions for use
- Dispose in accordance with national legal regulations, recycling indication
- Special warnings, always to comply with
- Packaging not suitable for contact with food
- Manufacturer

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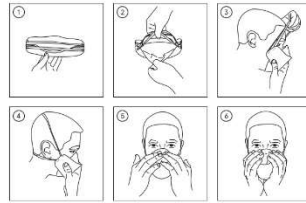
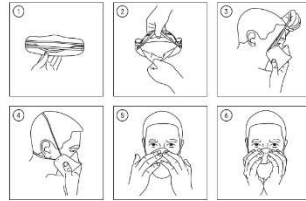
FITTING INSTRUCTIONS

Make sure your hands are clean and or wear clean gloves. Remove the mask from the packaging and check the components of the mask for damage before use.

1. Hold the mask with the broad side parallel to the body, the curved nose form points away from the body, the rubber straps upwards.
2. Take the rubber straps in one hand. With the thumb of the same hand, grasp the upper part of the mask with the nose clip. With the other hand, grasp the lower flap and completely unfold the mask to form a bowl.
3. Place the mask body (starting with the chin) over mouth and nose. Pull the rubber straps away from the mask and upwards over the head.
4. Make sure that the upper strap is placed over the upper back of the head and the lower strap under the ears over the lower back of the head and that the bands are not twisted. Align the upper and lower parts of the mask body so that the mask fits snugly, is comfortable and cannot slip during your activity. Make sure that no parts are folded inwards.
5. Use both hands to gently adjust the nose clip to the bridge and shape of your nose. Make sure that there is no gap above the bridge of the nose and that the mask fits tightly.
6. Inspect the correct fit of the mask before you start your activity (see FITTING INSPECTION).

FITTING INSPECTION

1. Carefully cover the mask body with both hands without changing its position on the face.
2. Exhale quickly and strongly.
3. If an airstream leaks out in the area of the bridge of the nose, readjust the nose clip and repeat steps 1 and 2 mentioned above.
4. If an airstream leaks out at the sealing line of the mask, check the fit of the straps on the head and adjust the position until no more air comes out at the edges. Repeat steps 1 and 2 mentioned above.

⚠ Do not enter a contaminated/dangerous area if you were not able to achieve a suitable fit.

STORAGE AND TRANSPORTATION

- The mask must be stored according to the manufacturer's instructions.
- Please use the original packaging for storing and transporting the mask.
- Do not expose the mask to direct sunlight or great heat.
- Please store the mask in a dry and frost-free environment.
- Temperature range of storage conditions: from +3°C to +33°C.
- Maximum relative humidity of storage conditions: 70%.
- Maximum shelf life of the mask is five years from the date of production (see batch imprint on packaging).

QUALITY AND QUALITY MANAGEMENT

PPERP X2 is approved as personal protection equipment (PPE) according to European Regulation (EU) 2016/425 and meets the requirements of EN 149:2001+A1:2009. Certificates, EU-Declaration of Conformity and additional information can be downloaded using the following link: www.pperp.de/downloads
Module C2 is assessed annually by: CCOS Certification Services Limited
Block 1 Blanchardstown Corporate Park,
Ballycoolin Road, Blanchardstown, D15 AKK1, Ireland
Number of the notified body: 2834